









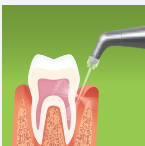








Application	Setting	Powder	Spray head	
 <p data-bbox="201 482 312 501">Supragingival</p>	 <p data-bbox="531 482 593 501">Prophy</p>	 <p data-bbox="821 393 906 505">Prophy</p>	 <p data-bbox="1096 482 1171 501">Standard</p>	 <p data-bbox="1334 482 1409 501">Optional</p>
 <p data-bbox="142 723 373 762">Supragingival, sulcus, subgingival pockets < 3 mm</p>	 <p data-bbox="539 723 585 742">Perio</p>	 <p data-bbox="821 628 906 740">Perio</p>	 <p data-bbox="1096 723 1171 742">Standard</p>	 <p data-bbox="1334 723 1409 742">Optional</p>
 <p data-bbox="142 969 373 988">Subgingival pockets > 3 mm</p>	 <p data-bbox="539 969 585 988">Perio</p>	 <p data-bbox="821 865 906 977">Perio</p>	 <p data-bbox="1096 969 1171 988">Standard</p>	 <p data-bbox="1334 969 1409 988">Optional</p>

#whdentalwerk



video.wh.com

Proxeo Aura

Air Polishing
System



Air polishing with **system**

Whether you are working supragingivally, in the sulcus or in deep pockets, the combination of the right powder, the corresponding setting on the adjustment ring and the matching spray head ensures you always achieve optimal treatment results.

proxeo AURA

20502 AEN Rev. 002 / 18.07.2022
Subject to alterations

Pictures are for illustrative purposes only. Additional equipment and accessories shown are not included as standard.

