OPERATION INSTRUCTIONS

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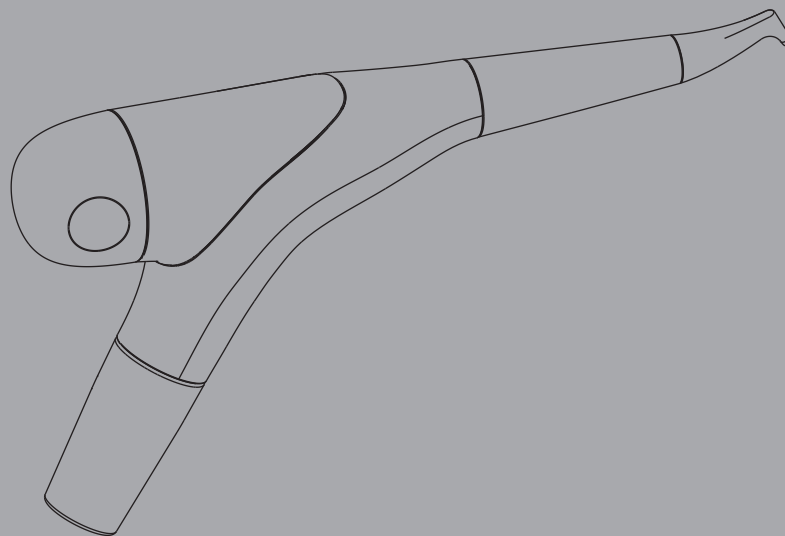
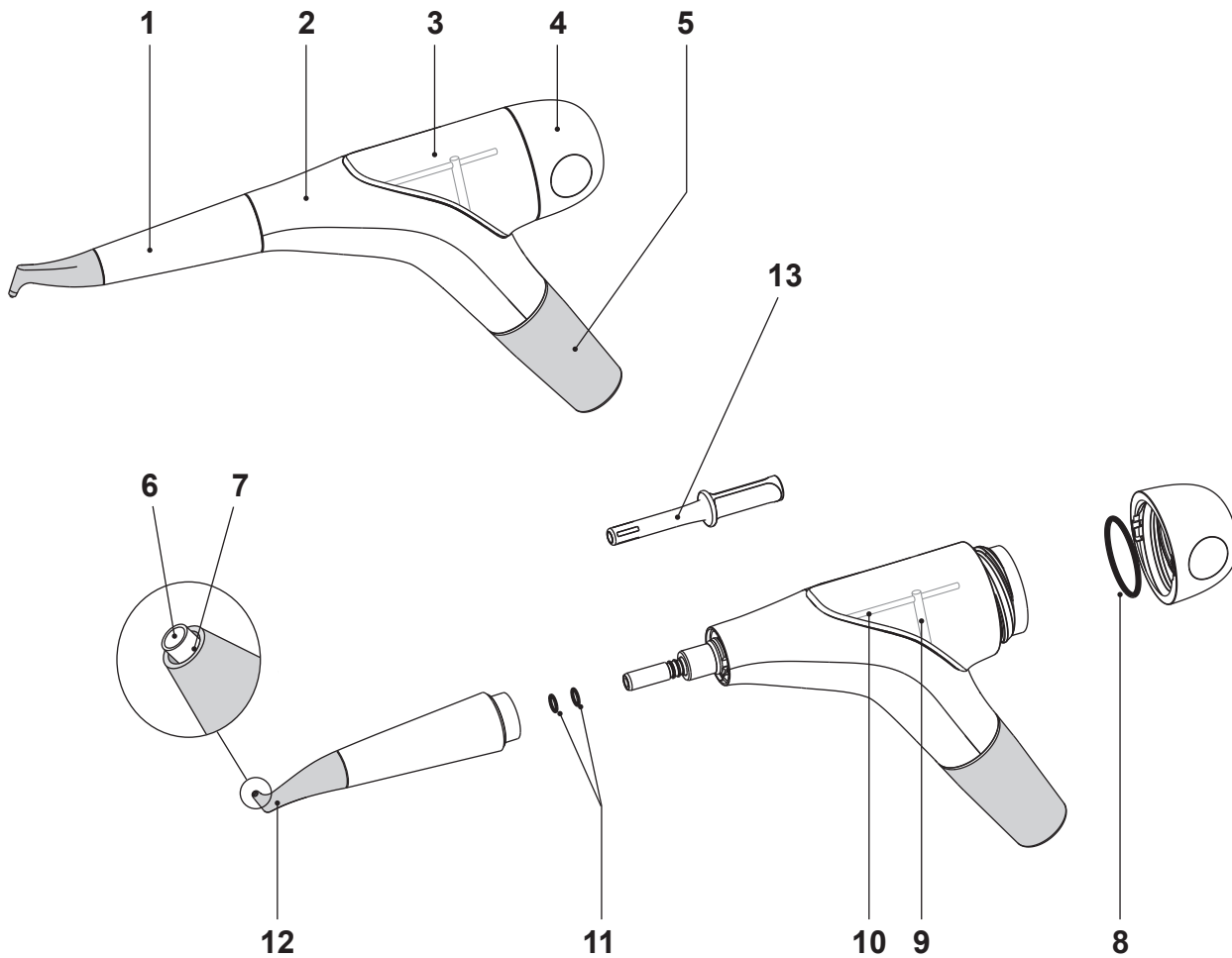
AIR - FLOW[®] handy 3.0

Rx ONLY Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

EMS⁺

COMPONENTS

- 1 Handpiece
- 2 Body
- 3 Powder chamber
- 4 Powder chamber cap
- 5 Cord adapter
- 6 Powder outlet tube
- 7 Water outlet port
- 8 Cap ring
- 9 Air tube
- 10 Air/powder tube
- 11 O-rings for handpiece connection
- 12 Nozzle
- 13 EasyClean

EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes.

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DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

Adaptable on the turbine connection of your dental unit, the air polishing AIR-FLOW handy 3.0 functions with the powders provided by EMS.

The confinement of the powder by a water jet makes it possible to steer the water jet with great precision and thus ensures a pleasant treatment for your patient.

This product removes dental plaque, soft deposits and surface stains from pits, grooves, interproximal spaces or smooth surfaces of the teeth.

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ABOUT THIS MANUAL

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

These operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all of the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close to hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger
Risk of injury

Caution
Risk of property damage or environmental harm

Please note
Useful additional information and hints

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CONTENTS

Treatment	2
Indications	2
Contra-indications	2
Installation and setting up	8
Water supply	8
Compressed air supply	8
Checking the turbine connection (Typical is shown)	10
Handpiece connection	12
Drying the sterilized handpiece	12
Connecting to the dental unit	14
Setting the water flow rate	16
Filling the powder chamber with EASY FILL	18
Closing the cap	20
Handling and water/air settings	22
Precautions	24
Wear a mask and eye protection	24
Treatment	28
Working Technique	30
Precautions to be taken at the end of the treatment	34
Fluoride Application	36
Patient Information	38
Start the cleaning	40
Cleaning, disinfecting and sterilizing	42
Cleaning the handpiece	42
Sterilization of the handpiece	44
Cleaning the unit	46
Daily cleaning of the device	48
Daily cleaning of the cap	50

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Maintenance.....	52
Replacing the cap and the O-rings.....	52
Annual maintenance	54
Safety precautions.....	56
Storing the product	60
Product disposal	62
Warranty.....	62
Accessories.....	62
EMS-Service.....	64
Symbols	66
Technical data.....	70
Troubleshooting	74

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TREATMENT

Indications

The AIR-FLOW handy 3.0 is recommended for :

- Removing stains/colorations and dental plaque
- Plaque removal for placement of sealants,
- Surface preparation prior to bonding/cementation of inlays onlays, crowns and veneers,
- Surface preparation prior to placing composite restorations,
- Cleaning prior to bonding orthodontic brackets,
- Effective plaque and stain removal for orthodontic patients,
- Stain removal for shade determination,
- Plaque removal prior to fluoride treatment,
- Plaque and stain removal prior to whitening procedure.

Contra-indications

 In order to avoid any risk of emphysema, hypersensitivity, bleeding and patient uncomfort, the use of supragingival powders (EMS CLASSIC, CLASSIC COMFORT and SOFT) is strictly forbidden during subgingival treatments (contacts with soft tissues, dentine, cementum).

 Patients suffering from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

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 Patients on a low salt diet must not be treated with a powder containing sodium bicarbonate. For patients on a low salt diet use the powders without sodium bicarbonate provided by EMS.

 Predisposed persons may be sensitive to the powder. If allergic reactions are observed, stop using the product and completely remove it.

 The aromatized powder provided by EMS can lead, in specific cases, to allergic reactions to the flavour. For patients known to suffer such reactions, use the powders without flavour provided by EMS.

 The powder containing sodium bicarbonate provided by EMS can be used only for supra-gingival application. For sub-gingival application, please use the specific powders provided by EMS and refer to their operation instructions.

 Do not direct the jet of powder toward fillings, or crown and bridgework as this could damage the restorations.

 Never use "EMS Abrasive Powder" for the treatments carried out with this product.

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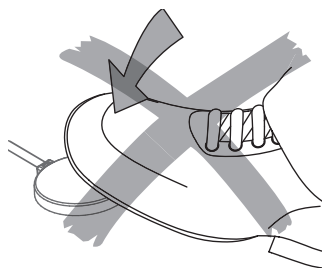
Checking the turbine connection (Typical is shown)

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⚠ The unit is equipped with an adapter specially designed for the turbine connection of your dental unit. Only use the unit with this special turbine connection. To connect it to another type of turbine will damage it.

⚠ The turbine of your dental unit must not be under pressure when to connect the unit. Do not activate the turbine footpedal. If you have a turbine equipped with light, turn the light off.

⚠ Make sure the o-rings of your turbine connection are in good condition. A turbine connection with o-rings in bad condition can damage the unit.

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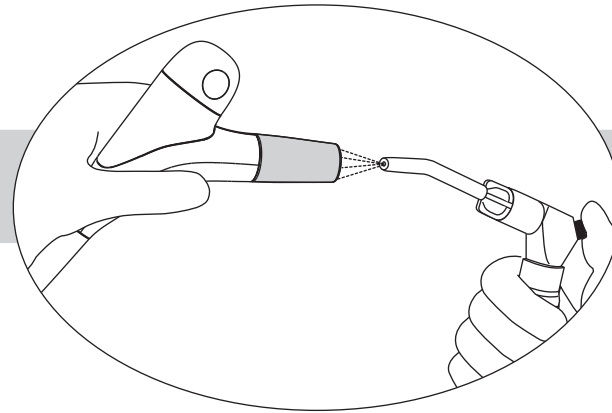
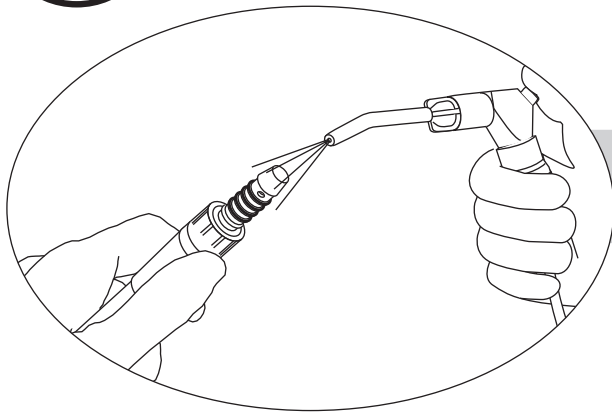
! The turbine connection and the connector must be absolutely dry. Moisture at the connection can cause blockage in the air/powder channels of the unit.

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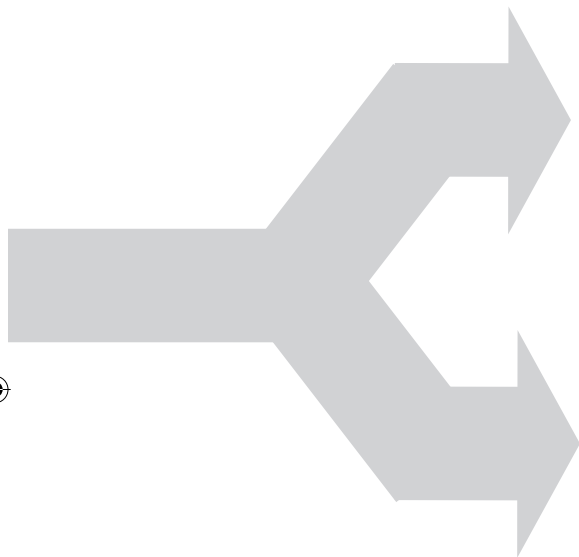
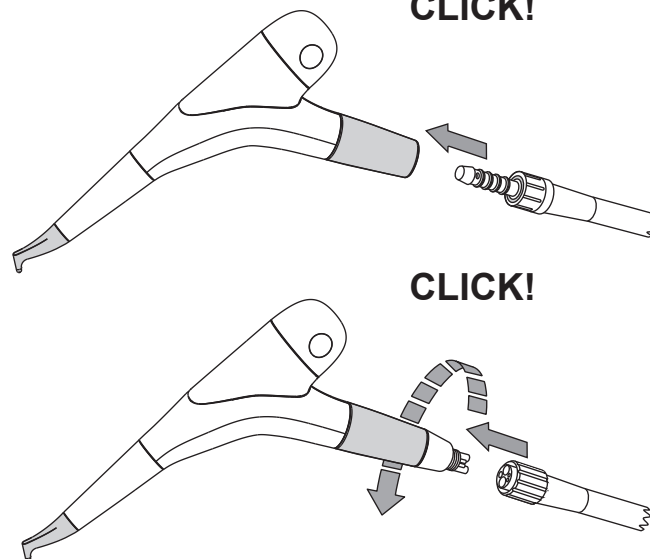
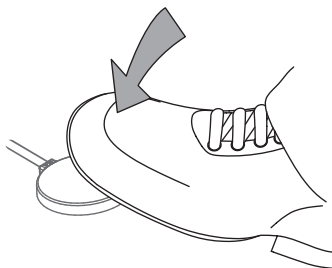
 Never use the handpiece without water flow.

 It is better to set the water flow rate before using the device for the first time while the powder chamber is empty.

Set the water flow rate from dental unit to obtain a small continuous spray.

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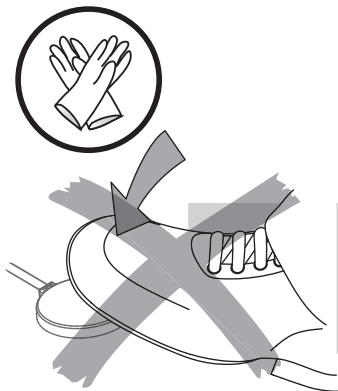
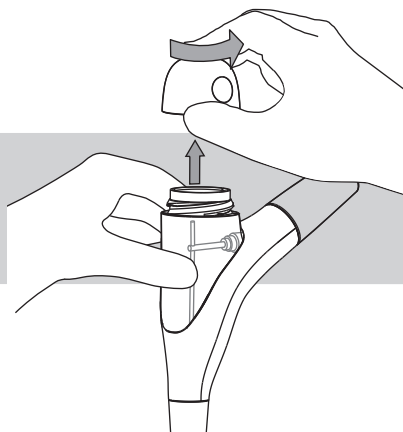
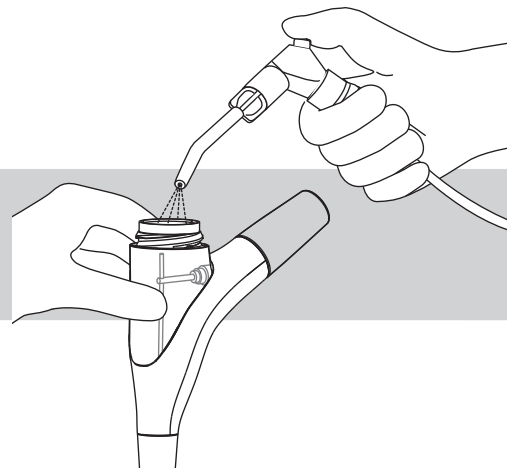
Filling the powder chamber with EASY FILL

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 Before using this product with the handpiece, please carefully read and follow the recommendations of the instruction manual originally delivered with this product.

 Make sure that the powder chamber and the cap of the powder chamber are absolutely dry. Moisture can cause the powder to cake.

 Use only powder provided by EMS.

 Do not go over the maximum limit as indicated on the picture above. The openings of the tubes should not be covered with powder. The tubes could get clogged.

 Close the bottle properly to preserve the powder of the humidity.

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 A powder jet accidentally directed into the eye can cause severe ocular injury. Therefore it is strongly recommended for all persons to wear eye protection during treatment.

 To limit the risk of bacterial or viral infections and inhalation of powder, we recommend to the dentist and the hygienist to wear gloves, protective glasses and a safety mask of protection.

 The contact lenses or glasses of the patient may get dirty during the treatment. We recommend to take them off.

 The nozzle should never be pointed toward the patient before, during and after operation.

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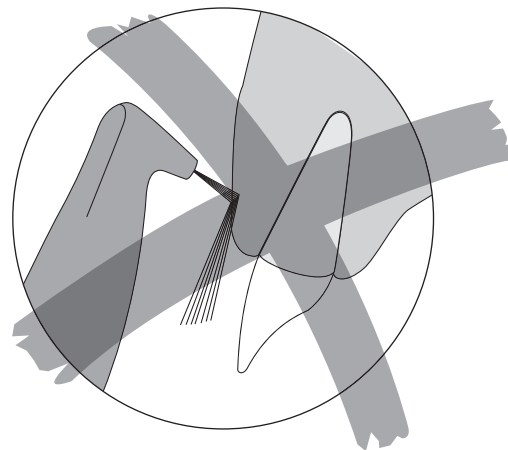
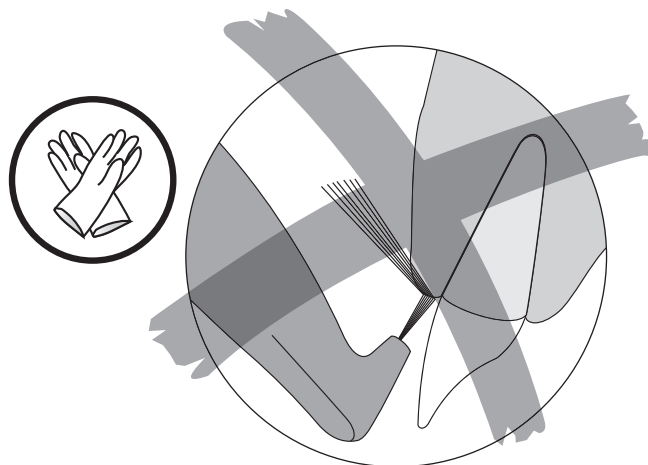
 The air/powder jet is powerful. It can cause injury to the gums or an emphysema caused by the introduction of air into the soft tissue spaces. It is strongly recommended that the operator never direct the nozzle directly at the gum tissue or into the gingival sulcus.

 During treatment, make small circular motions.

Finally after the treatment polish all dental surfaces by setting the water flow rate to the maximum.

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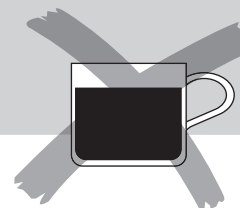
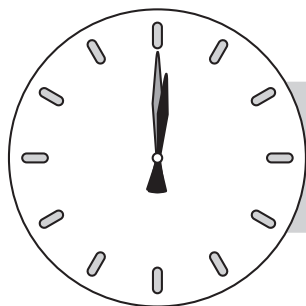
 After the treatment, the teeth are clean and the dental cuticle is completely eliminated. Its reconstitution by salivary proteins requires between 2 and 3 hours. During this time, the teeth do not have any more natural protection with respect to colouring.

 Tell your patient that during 2 to 3 hours following treatment, he should neither smoke, nor consume food or drinks which could strongly colour the teeth (tea, coffee...).

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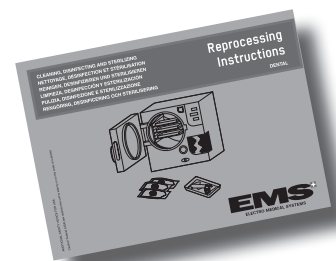
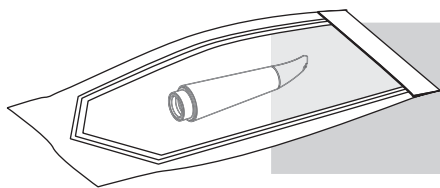
 Do not exceed the maximum number of sterilization cycles.

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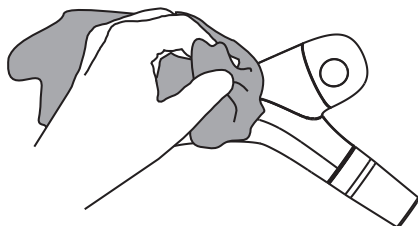
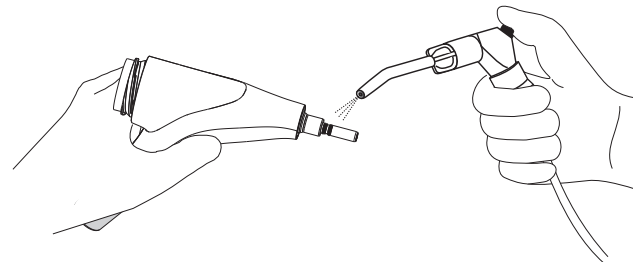
 The unit is not protected against water spray.

 Clean the unit only with an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant. The use of scouring powder or an abrasive sponge will damage its surface.

 Do not submerge the unit in a disinfectant bath. It would be likely to be damaged.

Make sure the handpiece connection of the cord is dry

 Do not sterilize the body and cord. It would be likely to be damaged.

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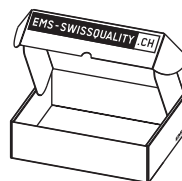
To keep your device in a perfect operating and to extend its life, send your device each year for an annual maintenance to EMS or an approved EMS repair center.

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SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operation instructions, or improper preparation and maintenance.

 Use for the intended purpose only. Before using the product, make sure that the manual has been read. This also applies to any equipment used with this product. Failure to observe the operation instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel (dentist or dental hygienist).

 Before each use, clean, disinfect and sterilize the different pieces and accessories of this product (nozzles excepted). Please refer to the information provided in the operation instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

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 In order to avoid the use of any inappropriate powders, it is highly recommended to always empty the powder chamber after use.

 The powders provided by EMS are especially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

 Never use "EMS Abrasive Powder" in the unit as this would damage it.

 Always examine the product for damage before starting the treatment. Damaged accessories or a damaged device must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

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 Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing at any time.

Should you wish to store the product for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing".
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data".

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PRODUCT DISPOSAL

The product shall not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

WARRANTY

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved EMS repair center.

In the case of non-authorized repairs or damaged due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

 Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operating instructions.

 The connector must be dismantled or changed only by an approved EMS repair center.

 You can ask for a modification of the type of adapter of the apparatus, in the range available near an approved EMS repair center only.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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TECHNICAL DATA

Description

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW handy 3.0
Classification 93 / 42 EEC	Class IIa
Mode	Continuous operation
Service Pressure to the turbine connection : Water	1 to 2.2 bar (1000-2200 hPa) with a service flow of 50-80 ml/min.
Service Pressure to the turbine connection : Air	Static pressure 2.7 to 3.5 bar (2700-3500 hPa)
Weight	Approx. 0.150 kg
Operating conditions	+10°C to +40°C 30% to 75% relative humidity 700 hPa to 1060 hPa air pressure Altitude max. 3000 m
Validated life time of the handpiece	800 sterilization cycles
Storage conditions	-10°C to +40°C 10% to 90% relative humidity 500 hPa to 1060 hPa air pressure
Transport conditions	-10°C to +40°C 10% to 90% relative humidity 500 hPa to 1060 hPa air pressure

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TROUBLESHOOTING

Type of problem	Solutions
Water comes into the powder chamber or leaks from the cap	<ul style="list-style-type: none">• Check the dental unit connection• Check the state of the o-rings of the turbine connection• Connect the unit to the dental unit (be careful, the powder chamber must be empty)• Dry and fill the powder chamber• Send the complete unit to an approved EMS repair center
No powder/air jet coming from the unit	<ul style="list-style-type: none">• Cut immediately the air input by releasing the foot pedal• Wait for 1-2 minutes for the system to decompress• Disconnect the unit from the dental unit• Press the foot control pedal of the dental unit• If no air comes out of the turbine connector, it is your dental unit that causes the problem• If air comes out of the turbine connector, it is the device that causes the problem• If air comes through the handpiece connector, the handpiece is blocked.• Clean the handpiece with the EasyClean• Unscrew the cap above the wash-hand basin, because the powder can be ejected.• A sealed apparatus, even disconnected, can indeed remain under pressure• Empty the powder chamber and screw the cap back on• Connect the unit to the dental unit (be careful, the powder chamber must be empty)• Press the foot control pedal of the dental unit• If no air comes out of the handpiece connector, the unit is blocked• Send the complete unit to an approved EMS repair center
Air and/or powder is leaking from the threads of the cap	<ul style="list-style-type: none">• Check the seal and the cleanliness of the thread on the powder chamber and on the cap• Replace the seal or the cap if necessary• Send the device to a certified EMS Repair Center
The unit efficiency diminishes	<ul style="list-style-type: none">• A powder refill is perhaps necessary• Clean the handpiece• Send the device to a certified EMS Repair Center
Water is leaking between the handpiece and the unit	<ul style="list-style-type: none">• Replace the seal of the handpiece connector• Send the complete unit to an approved EMS repair center
Air is leaking between the handpiece and the unit	<ul style="list-style-type: none">• Clean the handpiece• Replace the seal of the handpiece connector• Send the complete unit to an approved EMS repair center
Air and/or water is leaking from the connector of the cord	<ul style="list-style-type: none">• Check the connector of the cord, if it is necessary, carry out maintenance following the information of the manufacturer• Send the complete unit to an approved EMS repair center

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e-mail: info@ems-ch.de

SPAIN

EMS Electro Medical Systems España SL

Bernardino Obregón 14 bis

E-28012 Madrid

Tlf. +34 91 528 99 89

Fax +34 91 539 34 89

e-mail: administracion@ems-espana.com

FRANCE

EMS France Sarl

23, Av. Louis Bréguet

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F-78140 Vélizy Villacoublay

Tél. +33 1 34 58 03 80

Fax +33 1 34 58 03 90

e-mail: info@ems-france.fr

ITALY

EMS Italia S.r.l.

Via Faravelli 5

I-20149 Milano

Tel. +39 02 3453 8111

Fax +39 02 3453 2778

e-mail: dental@ems-italia.it

USA

EMS Corporation

11886 Greenville Avenue #120

Dallas, TX 75243, USA

Tel. +1 972 690 83 82

Fax +1 972 690 89 81

e-mail: info@ems-na.com

FB-568/5 Rev. A-03 23/01/2018

© Copyright EMS SA

EMS-DENTAL.COM