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INSTRUCTIONS FOR USE

Type of application:

Glycine-based prophylaxis powder for the gentle removal of supra- and subgingival plaque and discoloration. Suitable for all commercially available air polishing systems.

Notes on application:

Please observe the instructions for use and the maximum filling level of the device manufacturer.

Subgingival application:

- Removal of periodontal biofilms.
- Implant cleaning

• For supportive periodontal therapy (SPT) after initial treatment has been completed

Supragingival application:

- Removal of supragingival biofilms and discoloration around the sulcus
- Removal of discoloration in sensitive areas.
- Suitable for regular prophylaxis treatment

Never treat patients suffering from chronic bronchitis or asthma with an air polishing device. Do not use if sensitive to glycine. The patient and operator must wear eye protection. The operator must wear a mask and gloves. Use high-performance suction equipment. Apply Vaseline to the patient's lips. Do not direct the powder stream at fillings, crowns and bridges as these restorations could be damaged. Only for use in dentistry. Keep the container closed. **Shake well before use!**

Notes on incidents:

All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

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W&H Dentalwerk Bürmoos GmbH
PRISMA GmbH