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EN Orotol® plus Disinfection of Suction Systems

Orotol® plus is a highly effective aldehyde-free concentrate for the cleaning, disinfection, deodorization and care of dental suction systems as well as spittoon bowls, being likewise suitable for all amalgam separators.

Composition: 100 g Orotol® plus contain 8.8 g dimethyl-diocetyl-ammoniumchloride 50%, 1.2 g benzyl-dimethyl-dodecyl-ammoniumchloride 50%, 5 - 15 g alkaline cleansing agents, < 5 g complexing agents, special antifoaming agents, adjuvants.

Effective spectrum: Bactericidal, tuberculocidal, fungicidal, limited virucidal activity (encapsulated viruses incl. HBV, HCV, HIV⁴⁾ as well as non-encapsulated viruses, such as adeno viruses, noro viruses).

List of VAH. Tested in accordance with EN 13727, EN 13624, EN 14348, EN 14476, EN 14561, EN 14562, EN 14563, EN 16615.

¹⁾ As per RKI statement (Federal Health Bulletin 60, 353 - 363, 2017).

Directions for use: Orotol® plus is applied as a 2% solution (= 20 ml to 1 litre of water) using the OroCup. The exposition time is in accordance with VAH 5 min., for TB bacteria 60 min. (see table in DE). We recommend cleaning and disinfecting suction systems once or twice a day, according to work-load. Once-daily cleaning and disinfecting should be done at the end of the day. If necessary, clean and disinfect spittoon bowl with Orotol® plus, too. For further information please also see product information on the Internet.

Caution: After surgical procedures, aspirate a glass of cold water intermittently. No foaming agents must enter the suction system or the spittoon bowl.

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Danger. Peut être corrosive pour les métaux. Causes severe skin burns and eye damage. Harmful to aquatic life with long lasting effects. Wear protective gloves and eye/face protection. If skin irritation or rash occurs: Get medical advice/attention. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Rinse skin with water/shower. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

DÜRR DENTAL (Produkte) UK Ltd, 14 Linnei Way - Telford Way Industrial Estate Kettering, Northants NN16 6PS, United Kingdom ☎ +44 1536526740

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